



Oral Test [™] Instructions

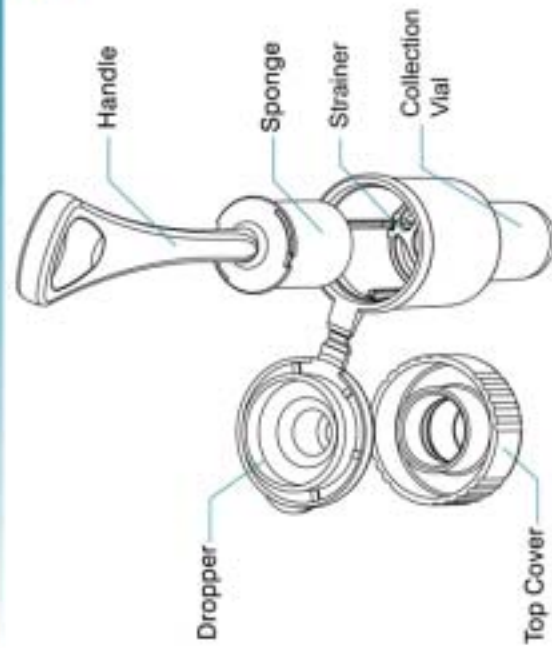


1 Hold the oral collector by its handle and insert the sponge into the mouth to collect the oral fluid. Swab the sponge continuously around the mouth. Press gently between tongue and the roof of the mouth to ensure full saturation. Collection is complete when there are no hard areas left and the collection sponge is saturated with oral fluid.




2 Flip open the top of the collection vial and insert the collector, sponge first. Push the collector down against the strainer to extract the oral fluid.

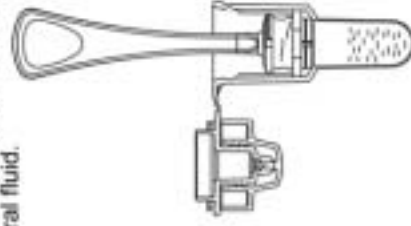


3 Close the top of the collection vial and unscrew the cap exposing the dropper.

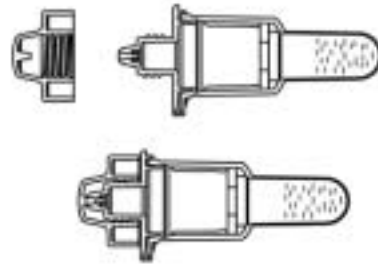


4 Gently squeeze the end chamber of the collection vial and dispense 3 drops into the sample well of the cassette. Replace the vial cap afterwards.

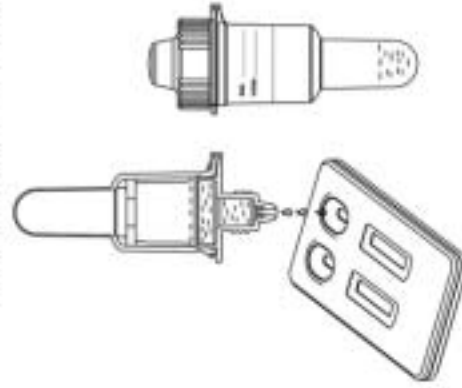

5 Read the results at 10 minutes.

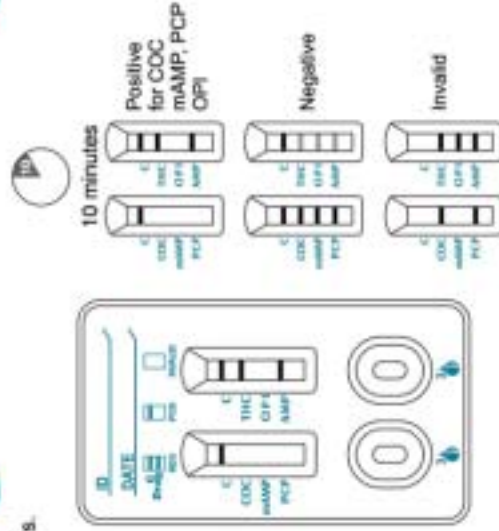

If you have a positive result, contact us for details of lab confirmation. You can then seal the vial and send to us at the address below

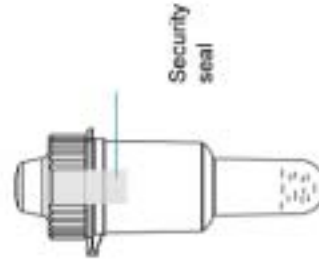

ORAL TEST PROCEEDURE

The Oral Test device is designed to be used with oral fluid only.

Preparation

Bring Test to room temperature before opening.

Ensure that the subject has had nothing in their mouth 5 minutes prior to conducting this test.

Unpack the components of the Oral Test and follow the instructions as shown overleaf.

Reading Results

You can read negative results as soon as the lines form, but for positive results please wait the full 10 minute development time.

Lab confirmation

If you have a positive result, the remaining fluid in the vial can be sent to us once you have contacted us for the details of confirmation service.

Drug Type	Code	Cutoff level
Amphetamines	AMP	50 ng/ml
Cocaine	COC	20 ng/ml
Methamphetamines	mAMP (MET)	50 ng/ml
Opiates	OPI	20 ng/ml
Cannabis	THC	12 ng/ml
Phencyclidine	PCP	10 ng/ml

For any technical queries please call our technical staff on the number below.



SureScreen Diagnostics Ltd
1 Prime Parkway
Derby
DE1 3QB
UK

T: +44 (0)1332 365318
F: +44 (0)1332 292230
E: info@surescreen.com
technical@surescreen.com
www.surescreen.com